

POLYPROPYLENE HOLLOWWARE **CLEANING & DISPOSAL GUIDANCE**

Henleys polypropylene hollowware products have a softening temperature of 130°C.

Washer Disinfectors

Henleys polypropylene products can be cleaned in washer disinfectors to 90°C. Follow current guidelines and your washer disinfectant instructions.

Wipes

Henleys polypropylene products can be cleaned using wipes designed for plastics, including Clinell Sporidical or Universal Wipes.

Disposal

Polypropylene is classified as a Class 5 recyclable plastic material. Hospital waste products must be disposed of in accordance with regulations on clinical waste and hospital protocol. Polypropylene is suitable for incineration.

Information correct as of September 2024
Y:0024 03

Registered Office:

HENLEYS MEDICAL SUPPLIES LIMITED

Brownfields, Welwyn Garden City, Herts AL7 1AN

Tel: 01707 333164 Fax: 01707 334795

www.henleysmed.com

Registered in England No, 0452882